wiroSKIP

[de]	Produktdatenblatt PP ÜBERSCHUHE	Seite 2
[en]	Product data sheet PP OVERSHOES	Page 5
[nl]	Productgegevensformulier SCHOENOVERTREKKEN	Zijde 8
[fr]	Fiche produit SURCHAUSSURES EN PP	Page 11
[es]	Hoja de datos del producto CUBRECALZADOS DE PP	Página 14
[it]	Scheda prodotto SOVRASCARPE IN PP	Pagina 17
[el]	Δελτίο δεδομένων προϊόντος ΠΟΔΟΝΑΡΙΑ PP	Σελίδα 20
[tr]	Ürün Bilgi Formu PP GALOŞLAR	Sayfa 24
[ru]	Техническая спецификация БАХИЛЫ ПОЛИПРОПИЛЕНОВЫЕ (PP)	Сторона 27
[pl]	Karta produktu OCHRONIACZE NA BUTY PP	Strona 30
[da]	Produktdatablad PP-OVERTRÆKSSKO	Side 33
[sv]	Produktdatablad PP SKOÖVERDRAG	Sida 36
[no]	Produktdatablad PP-SKOTREKK	Side 39
[cs]	Informační list výrobku PP NÁVLEKY NA OBUV	Strana 42
[hu]	Termékadatlap PP CIPŐVÉDŐK	Oldal: 45
[sk]	Informačný list výrobku PP NÁVLEKY	Strana 48
[sl]	Podatkovni list izdelka PP PREVLEKE ZA OBUTEV	Stran 51
[ro]	Fișa produsului BOTOȘI DIN PP	Pagina 54
[pt]	Folha de dados do produto PROTETORES DE CALÇADO EM PP	Página 57
[lt]	Gaminio duomenų lapas PP ANTBAČIAI	Puslapis 60
[lv]	Produkta datu lapa PP APAVU PĀRVALKI	Lappuse 63
[bg]	Информационен лист за продукта PP КАЛЦУНИ	Страница 66
[uk]	Технічний лист БАХІЛИ З ПОЛІПРОПІЛЕНУ	Сторінка 69
[et]	Toote teabeleht PP-KILESUSSID	Lk 72
[fi]	Tuotetietolehti PP-KENGÄNSUOJAT	Sivu 75

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PP OVERSHOES

Anti-slip rubber coating on the sole

PREMIUM

Product data sheet

wiroSKIP

Manufacturer	
Company + Address	WIROS Wilfried Rosbach GmbH, Karl-Arnold-Str. 5, 47877 Willich, Germany
Trademark	care & serve®
Product description	
Category	PREMIUM
Product description	care & serve® PREMIUM wirosSKIP PP overshoes Anti-slip rubber coating on the sole
Product features	Anti-slip rubber coating on the sole
Material	
Material	Non-woven polypropylene
Material weight	≈ 35 g/m²
Application	
Intended use	Disposable overshoes to cover shoes for protection from contamination or to keep floors clean.
Contact with food produce	Short-term indirect contact with foodstuffs is permitted.
Notes on use	Disposable product
Limitation	Not suitable for packing or storing food.
Application areas	Industry Pharmaceutical industry HACCP House + Garden Agriculture Construction Industry
Supplementary material info	Latex-free
Health advice	If skin irritation or allergic reactions occur, discontinue use immediately and seek medical advice. As far as is known according to the current state of the art, the products do not contain any toxic, carcinogenic or mutagenic substances when used in accordance with their intended use. These products do not contain latex components and are considered hypoallergenic. However, they can cause irritation in sensitive skin. Long-term wear may restrict skin ventilation, which can cause skin irritation.
Standards	
VO (EU) 2023/988	Regulation (EU) No 2023/988 on general product safety
VO (EG) 1935/2004	Regulation regarding plastic materials and articles intended to come into contact with food.
VO (EU) 10/2011	Regulation on plastic materials and articles intended to come into contact with food

Product data sheet

wiroSKIP

Use	
Instructions for use	Before use Remove the disposable overshoes from the packaging and check them for damage or defects. Make sure that they are clean and hygienic. Use Place your foot into the disposable overshoe and pull it over your shoe. Ensure that the overshoe covers your entire shoe and fits well. The disposable overshoes are elastic and adapt to the shape of your shoes. Make sure that the overshoes fit well on your shoes and do not slip. To take them off, pull the disposable overshoes carefully off your shoes and dispose of them securely. Avoid shaking or touching the overshoes when you take them off in order to minimise the soiling of your shoes. It is important to note that disposable overshoes primarily serve to protect your shoes from soiling and at the same time to ensure hygiene in certain work areas.
Safety notice	According to our risk analysis under EN ISO 14971:2019 + A11:2021, no additional safety instructions are required for this product.
Disposal	Unused and uncontaminated products can be incinerated in an environmentally friendly manner or disposed of at a waste disposal site. Contaminated products must be disposed of in accordance with the national laws and regulations valid for the respective type of contamination.
Shelf life	5 years from the production date, if unused and stored correctly.
Storage	Keep out of direct sunlight. Store in a dry place. Incorrect storage can lead to a reduction in the shelf life. 
Traceability	Batch-related traceability from the determination of requirements to issuing. Product data sheet Version 1 Valid from Batch 2617 +

Packaging	
Recycling	Licensed in accordance with VerpackG (German Packaging Act), Licence number DE1618662887322. Please recycle the packaging.  \$VerpackG

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The numerical data for the measurements, weights, thicknesses and tolerances are target values, and there may be some deviation.

The information represents the current state of our knowledge. It offers no guarantee of the product characteristics and does not form the basis of a contractual legal understanding. Please reconfirm any information that is older than 2 years.

Typographical and other errors excepted.

The product data sheets provided are the property of WIROS Wilfried Rosbach GmbH. WIROS Wilfried Rosbach GmbH is the owner of all copyrights and rights of use and exploitation for the product data sheets. It is not permitted to change the product data sheets or to use the pictograms. The product data sheets provided may only be used in conjunction with original WIROS products.

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