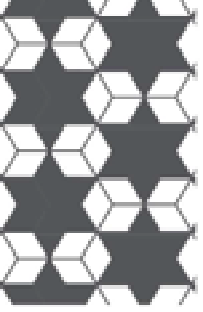
wiroSTRUCTUREGRIP PF

[de]	Produktdatenblatt NITRIL HANDSCHUHE	Seite 2
[en]	Product data sheet NITRILE GLOVES	Page 8
[nl]	Productgegevensformulier NITRIL HANDSCHOENEN	Zijde 14
[fr]	Fiche produit GANTS EN NITRILE	Page 20
[es]	Hoja de datos del producto GUANTES DE NITRILO	Página 26
[it]	Scheda prodotto GUANTI IN NITRILE	Pagina 32
[el]	Δελτίο δεδομένων προϊόντος ΓΑΝΤΙΑ ΝΙΤΡΙΛΙΟΥ	Σελίδα 38
[tr]	Ürün Bilgi Formu NITRIL ELDIVENLER	Sayfa 43
[ru]	Техническая спецификация ПЕРЧАТКИ НИТРИЛОВЫЕ	Страница 49
[pl]	Karta produktu RĘKAWICE NITRYLOWE	Strona 55
[da]	Produktdatablad NITRIL-HANDSKER	Side 61
[sv]	Produktdatablad NITRILHANDSKAR	Sida 666
[no]	Produktdatablad NITRIL-HANSKER	Side 71
[cs]	Informační list výrobku NITRILOVÉ RUKAVICE	Strana 76
[hu]	Termékadatlap NITRIL VÉDŐKESZTYŰ	Oldal: 81
[sk]	Informačný list výrobku NITRILOVÉ RUKAVICE	Strana 86
[sl]	Podatkovni list izdelka ROKAVICE IZ NITRILA	Stran 91
[ro]	Fișa produsului DEGETARE DIN NITRIL	Pagina 96
[pt]	Folha de dados do produto LUVAS DE BORRACHA NITRÍLICA	Página 102
[lt]	Gaminio duomenų lapas NITRILO PIRŠTINĖS	Puslapis 107
[lv]	Produkta datu lapa NITRILA CIMDI	Lappuse 112
[bg]	Информационен лист за продукта НИТРИЛНИ РЪКАВИЦИ	Страница 117
[uk]	Технічний лист НІТРИЛОВІ РУКАВИЧКИ	Сторінка 122
[et]	Toote teabeleht NITRIILKINDAD	Lk 128
[fi]	Tuotetietolehti NITRIILIKÄSINEET	Sivu 133

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wiroSTRUCTUREGRIP PF

NITRIL HANDSCHUHE

Puderfrei | Rauten-Grip-Textur

PROFESSIONAL

Hersteller

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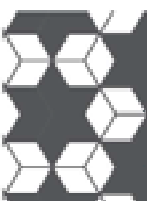
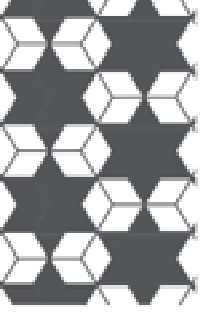
wiroSTRUCTUREGRIP PF

NITRILE GLOVES

Powder-free | Diamond grip structure

PROFESSIONAL

Product data sheet

wiroSTRUCTUREGRIP PF

Manufacturer

Company + Address	WIROS Wilfried Rosbach GmbH, Karl-Arnold-Str. 5, 47877 Willich, Germany
Trademark	care & serve®

Product description

Category	PROFESSIONAL
Product description	care & serve® PROFESSIONAL wiroSTRUCTUREGRIP PF Nitrile gloves Powder-free Diamond grip structure
Product features	Powder free Diamond grip texture

Material

Material	Synthetic nitrile rubber
Thickness	Finger: ≈ 0.23 mm (± 0.01 mm) Palm: ≈ 0.19 mm (± 0.01 mm) Hand edge: ≈ 0.10 mm (± 0.01 mm)
Material weight	Size M ≈ 6.5 g/pc. (± 0.2 g)


Application

Intended use	Disposable protective gloves to prevent the surface of the hands from coming into contact with substances, mixtures, or harmful biological agents that are hazardous to health. The level of protection and the maximum wearing time come from the determined performance level for each material tested.
Contact with food produce	Short-term indirect contact with foodstuffs is permitted.
Notes on use	Disposable product Not sterile
Limitation	It is the sole responsibility of the user to determine the suitability and/or duration of use of care & serve® products for their intended use. The manufacturer assumes no responsibility for the improper use of care & serve® products. If a risk analysis has deemed that the protection listed above is NOT sufficient, protective clothing with higher protective properties should be used.
Application areas	Industry Pharmaceutical industry HACCP Cleaning Hygiene House + Garden Agriculture Construction Industry
Supplementary material info	Latex-free
Health advice	If skin irritation or allergic reactions occur, discontinue use immediately and seek medical advice. As far as is known according to the current state of the art, the products do not contain any toxic, carcinogenic or mutagenic substances when used in accordance with their intended use. Contains no latex proteins and is therefore suitable for people with latex allergies. However, may contain vulcanisation accelerators (type IV allergens).

Standards

CE marking	Personal protective equipment category III in accordance with Regulation (EU) 2016/425
------------	--

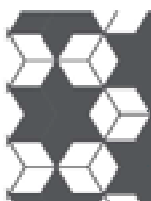wiroSTRUCTUREGRIP PF

CE marking

The CE mark guarantees the free movement of products and goods within the European Economic Community. All products with the CE mark comply with the essential requirements of the European regulations specified.

EN ISO 21420:2020

Standard: Protective gloves - General requirements and test methods

Test	Result
pH value	Passed
Mobility	5
Minimum length	Length
Size 7: 230 mm	≈ 240 mm
Size 8: 240 mm	≈ 240 mm
Size 9: 250 mm	≈ 240 mm
Size 10: 260 mm	≈ 240 mm
Size 11: 270 mm	≈ 240 mm

Information

Observe the product information and instructions for use.

EN 455-1:2022

Medical gloves for single use - Part 1: Requirements and testing for freedom from holes

Test	Result
Water retention test	Passed
Seal tightness	AQL 1.5

EN 455-2:2015

Medical gloves for single use - Part 2: Requirements and testing for physical properties

Test	Result
Minimum length	Length
XS - XL ≈ 240 mm	≈ 240 mm
Tensile strength	Result
Tensile strength	Passed

EN 455-3:2015

Medical gloves for single use - Part 3: Requirements and testing for biological evaluation

Test	Result
Powder residues	0,11 mg/g

EN 455-4:2009

Medical gloves for single use - Part 4: Requirements and testing for shelf life determination

Test	Years
Shelf life	5

EN ISO 374-1:2016 + A1:2018 TYPE B

Standard: Protective gloves against dangerous chemicals and micro-organisms - Part 1: Terminology and performance requirements for chemical risks.

Test chemical	Level
J n-Heptane	2
K Sodium Hydroxide 40%	6
L Sulphuric Acid 96%	1
O Ammonium Hydroxide 25%	6
P Hydrogen Peroxide 30 %	4
T Formaldehyde 37%	5

Information

Breakthrough time: Level 1 > 10 min. | Level 2 > 30 min. | Level 3 > 60 min. | Level 4 > 120 min. | Level 5 > 240 min. | Level 6 > 480 min.

This information does not specify the actual protection duration in the workplace and does not distinguish between mixtures and pure chemicals.

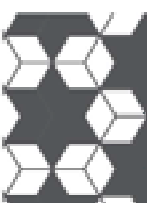
The resistance to chemicals is assessed under laboratory conditions using samples only taken from the palm of the hand (except where the glove measures 400 mm or longer - in this case, the cuff is also tested), and applies only to the tested chemicals. The resistance can alter when the chemicals are used in a mixture.

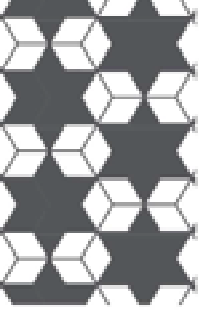
We recommend checking whether the gloves are suitable for the intended application, as the conditions at the workplace can vary from those in the type test depending on the temperature, abrasion and degradation.

If protective gloves are already in use, these can offer reduced resistance to dangerous chemicals due to changes in their physical properties. Degradation caused by contact with chemicals, movements pulling threads, rubbing, etc. can considerably reduce the actual duration of use. Degradation can be the most important factor with aggressive chemicals, which must be taken into consideration when selecting chemical-resistant gloves.

The gloves must be checked for flaws or defects before use.

For single use only, do not reuse.

Product data sheet

wiroSTRUCTUREGRIP PF


EN ISO 374-2:2019	Protective gloves against dangerous chemicals and microorganisms - Part 2: Determination of resistance to penetration															
	<table border="1"> <thead> <tr> <th>Test</th> <th>Result</th> </tr> </thead> <tbody> <tr> <td>Water penetration test</td> <td>Passed</td> </tr> </tbody> </table>	Test	Result	Water penetration test	Passed											
Test	Result															
Water penetration test	Passed															
EN ISO 374-4:2019	Protective gloves against dangerous chemicals and microorganisms - Part 4: Determination of resistance to degradation by chemicals															
	<table border="1"> <thead> <tr> <th>Test chemical</th> <th>Degradation %</th> </tr> </thead> <tbody> <tr> <td>J n-Heptane</td> <td>50,6</td> </tr> <tr> <td>K Sodium Hydroxide 40%</td> <td>-7,1</td> </tr> <tr> <td>L Sulphuric Acid 96%</td> <td>100</td> </tr> <tr> <td>O Ammonium Hydroxide 25%</td> <td>27</td> </tr> <tr> <td>P Hydrogen Peroxide 30 %</td> <td>27,1</td> </tr> <tr> <td>T Formaldehyde 37%</td> <td>17,7</td> </tr> </tbody> </table>	Test chemical	Degradation %	J n-Heptane	50,6	K Sodium Hydroxide 40%	-7,1	L Sulphuric Acid 96%	100	O Ammonium Hydroxide 25%	27	P Hydrogen Peroxide 30 %	27,1	T Formaldehyde 37%	17,7	
Test chemical	Degradation %															
J n-Heptane	50,6															
K Sodium Hydroxide 40%	-7,1															
L Sulphuric Acid 96%	100															
O Ammonium Hydroxide 25%	27															
P Hydrogen Peroxide 30 %	27,1															
T Formaldehyde 37%	17,7															
EN ISO 374-5:2016 VIRUS	Standard: Protective gloves against dangerous chemicals and micro-organisms - Part 5: Terminology and performance requirements for micro-organism risks															
	<table border="1"> <thead> <tr> <th>Test</th> <th>Result</th> </tr> </thead> <tbody> <tr> <td>Protection against bacteria and fungi</td> <td>Passed</td> </tr> <tr> <td>Protection against bacteria, fungi and viruses</td> <td>Passed</td> </tr> </tbody> </table>	Test	Result	Protection against bacteria and fungi	Passed	Protection against bacteria, fungi and viruses	Passed									
Test	Result															
Protection against bacteria and fungi	Passed															
Protection against bacteria, fungi and viruses	Passed															
	Information The penetration resistance is assessed under laboratory conditions and refers solely to the tested samples.															
VO (EU) 2023/988	Regulation (EU) No 2023/988 on general product safety															
VO (EG) 1935/2004	Regulation regarding plastic materials and articles intended to come into contact with food.															
VO (EU) 10/2011	Regulation on plastic materials and articles intended to come into contact with food															
XXI/1.	Recommendation of the Federal Institute for Risk Assessment XXI/1. on commodities based on natural and synthetic rubber in contact with food															
	Information Category 3 - Short-term contact															
	Only authorised substances according to Table 1 BfR recommendation XXI/1, additives according to BfR recommendation XXI and Table 2 of BfR recommendation XXI/1 are used. The recommended limit values are upheld.															

Use

Instructions for use	<p>Before use</p> <p>Please make sure that you have selected the correct product for your requirements and application areas, and observe the requirements set out in the respective standards. Should you have any questions, please consult the supplier of your disposable protective gloves.</p> <p>Please do not wear the disposable protective gloves if they show visible defects or have already been used. Disposable protective gloves are only intended for short-term use when working with mild chemicals.</p> <p>Use</p> <p>Please select the correct size for you.</p> <p>The disposable protective gloves protect the area they cover. If protection is required for a different part of the body or the respiratory system, add additional PPE products and, for each application, only use the PPE approved for that intended purpose. The user is solely responsible for the correct combination of additional PPE and compliance with the necessary requirements of the standards for each application.</p> <p>You alone are responsible for the duration of use during the respective application, while taking into consideration the protective effect and wear comfort. An alteration in the given protection level cannot be excluded following contamination. Also check regularly during use that the item is intact. Replace a damaged disposable protective glove with an intact one immediately. Remove the disposable protective glove if it is contaminated or damaged and dispose of it correctly.</p>
Safety notice	According to our risk analysis under EN ISO 14971:2019 + A11:2021, no additional safety instructions are required for this product.
Disposal	Unused and uncontaminated products can be incinerated in an environmentally friendly manner or disposed of at a waste disposal site. Contaminated products must be disposed of in accordance with the national laws and regulations valid for the respective type of contamination.
Shelf life	5 years from the production date, if unused and stored correctly.

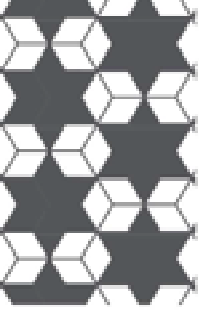
Product data sheet

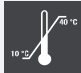wiroSTRUCTUREGRIP PF

Storage	Protect from intense heat, sunlight, frost, moisture, ozone, X-rays and ultraviolet light. Store in a dry place. Storage temperature between 10 °C and 40 °C. Incorrect storage can lead to a reduction in the shelf life.	  
Traceability	Batch-related traceability from the determination of requirements to issuing. Product data sheet Version 2 Valid from Batch 2617 +	

Packaging

Recycling	Licensed in accordance with VerpackG (German Packaging Act), Licence number DE1618662887322. Please recycle the packaging.	
------------------	---	---

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

Variant 1

Sales unit	1000 Unit/Cardboard box
Packaging level 1	100 Unit/Dispenser box
Minimum order quantity	1 Cardboard box

Item numbers	Item numbers				
	Item number	Colour	Size	Size (W x L)	Tolerance
	40458771	black	S	≈ 80 x 240 mm	± 10 mm
	40458772	black	M	≈ 95 x 240 mm	± 10 mm
	40458773	black	L	≈ 110 x 240 mm	± 10 mm
	40458774	black	XL	≈ 120 x 240 mm	± 10 mm
	40458775	black	XXL	≈ 130 x 240 mm	± 10 mm
	40458781	orange	S	≈ 80 x 240 mm	± 10 mm
	40458782	orange	M	≈ 95 x 240 mm	± 10 mm
	40458783	orange	L	≈ 110 x 240 mm	± 10 mm
	40458784	orange	XL	≈ 120 x 240 mm	± 10 mm
	40458785	orange	XXL	≈ 130 x 240 mm	± 10 mm

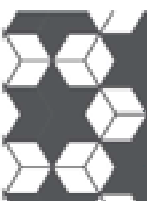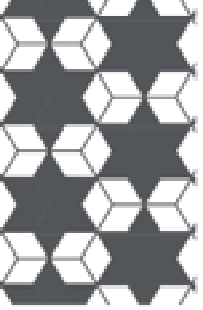Product data sheet


wiroSTRUCTUREGRIP PF


GTINs	Item number	Unit of Use	Packaging level 1	Packaging level 2	Cardboard box
		UDI	UDI	UDI	UDI
	40458771	-----	4051642010656	-----	4051642010649
	40458772	-----	4051642010670	-----	4051642010663
	40458773	-----	4051642010694	-----	4051642010687
	40458774	-----	4051642010717	-----	4051642010700
	40458775	-----	4051642010854	-----	4051642010847
	40458781	-----	4051642010731	-----	4051642010724
	40458782	-----	4051642010755	-----	4051642010748
	40458783	-----	4051642010779	-----	4051642010762
	40458784	-----	4051642010793	-----	4051642010786
	40458785	-----	4051642010830	-----	4051642010823

The numerical data for the measurements, weights, thicknesses and tolerances are target values, and there may be some deviation.

The information represents the current state of our knowledge. It offers no guarantee of the product characteristics and does not form the basis of a contractual legal understanding. Please reconfirm any information that is older than 2 years. Typographical and other errors excepted.

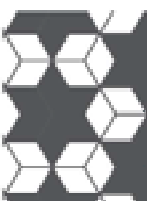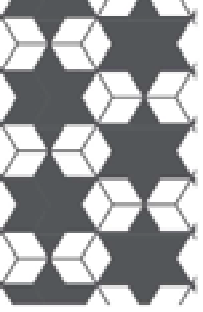The product data sheets provided are the property of WIROS Wilfried Rosbach GmbH. WIROS Wilfried Rosbach GmbH is the owner of all copyrights and rights of use and exploitation for the product data sheets. It is not permitted to change the product data sheets or to use the pictograms. The product data sheets provided may only be used in conjunction with original WIROS products.

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